www.relegs.com

SNOOZE YOUR RLS

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Foreword

This manual describes Relegs®; a device for treating restless legs syndrome (RLS). In accordance with the Law on Medical Devices, Relegs® is classified as a class IIa medical device. This manual provides information about safety aspects, a description of Relegs® and the working principle, as well as instructions, operation and maintenance. The manual also describes the potential hazards and instructions to prevent such hazards. It is important that you read this manual thoroughly to learn how Relegs® should be operated and how it should be maintained.

Reading and following the instructions in this manual will help you to use Relegs® correctly, so that you can prevent personal injury and damage to the device. Relegs B.V. pays considerable attention to safety.

The user is responsible for the correct use and for ensuring timely maintenance of the device.

Relegs BV.
Kapteynstraat 1
2201 BB Noordwijk
the Netherlands
service@relegs.com

RELEGS BV reserves the right to correct or modify this manual at any time and without prior notification. However, changes can be published in new editions of this manual.

Table of contents

Foreword	46
1. What the symbols mean.....	48
2. Product description.....	49
3. Safety.....	50
4. Contens of the packaging.....	55
5. Charging the battery	56
6. Maintenance and Cleaning.....	57
7. Disposal	58
8. Instructions for use of Relegs®.....	59
9. Explanation of indicator lights.....	62
10. Technical malfunction.....	63
11. Electromagnetic immunity.....	64
12. Technical speficiations.....	65
13. Limited warranty	67

1. What the symbols mean

The safety symbols used in this manual serve as instructions for the correct use of Relegs® and to protect your safety.

SYMBOL	MEANING
	Warning/danger! Incorrect use can result in a risk of serious injury, damage and even danger to life. You must absolutely follow these instructions!
	Warning/danger! the device may not be used by people with a pacemaker.
 Li-ion	Contains a Lithium battery.
	Symbol for separate collection of electrical and electronic equipment.
	Applied part of type F in accordance with IEC 60601-1, which offers a higher protection against electrical shock.

2. Product description

Relegs® is a wireless miniature electrostimulator designed to be attached to the calf via an electrode patch.

Relegs® is easy to operate by tapping on the device. It can be used under your clothing. Relegs® has a unique peel-off detection system for safe use in bed.

2.1. INTENDED USE

Relegs® is intended for use by people with Restless Legs Syndrome (RLS). Relegs® can be used as a stand-alone treatment for RLS or in addition to other treatments, including medication.

RLS is a common neurological condition. The exact cause is not known yet. The condition is characterized by a restless feeling in the legs and the irresistible urge to move the legs. Symptoms of RLS occur in rest, when you are lying in bed and want to go to sleep, or when you sit down for an extended time; at home on the couch, at work, during an evening out (movie, theatre), travelling in a car, bus or airplane. Relegs® is easy to use in all situations where RLS occurs.

2.2. WORKING PRINCIPLE

Relegs® emits harmless electrical pulses, which are transmitted through the skin via an electrode patch to stimulate the nerves. This creates contra stimulation in the calf and provides immediate relief of the restless legs and reduces the urge to need to move the legs. Select the highest intensity that you are able to tolerate with your pain threshold to achieve a positive effect. This should produce a strong, but still comfortable, sensation. Relegs® does not treat the cause of RLS.

3. Safety

3.1. GENERAL SAFETY INSTRUCTIONS

- If the Relegs® system is defective, you should never attempt to repair, use or modify the device yourself. Incorrect use can result in pain, injury or burns from the electrostimulation.
- Stop using Relegs® immediately and consult your physician if you experience changes to the skin, pain, swelling or other anomalies during use.
- Stop using the device immediately if any anomalies of the device occur during use. Please contact the service centre (service@relegs.com) if the device is not working properly.
- Keep this user manual for the lifespan of this device, in case you have questions at a later stage and include this manual when passing the device on to third parties. Also make the user manual available to third parties. The user manual forms part of the Relegs® system.
- Avoid misuse and any use other than for which the device was intended.
- Use Relegs® exclusively with the electrode patches that are supplied with the device, which can be reordered via the website www.relegs.com. Other electrodes can have electrical properties that are not adapted to the stimulator.
- Avoid contact between the electrode patch and metal objects located on your body, (e.g. jewelry, belt, watch).
- All serious incidents that occur with use of Relegs® must be reported to Relegs B.V. (service@relegs.com) and the competent authority of the member state in which the user and/or patient is located.

3.2. APPLICATION RANGE RELEGS®

- Relegs® is solely intended for external use (application on the skin) in humans, with the aim of electrically stimulating the nerves.
- Treatment of RLS with Relegs® is purely symptomatic in nature. This can provide relief of RLS symptoms. Please consult your physician if you have therapeutic questions.
- Relegs® has a pre-set treatment time of 30 minutes. The treatment time can be shortened or extended, depending on the severity of the RLS.
- The perception of the intensity definitely depends on the person's state of mind on any given day. The intensity can be adjusted according to individual needs at that time.
- Sometimes a person will experience cramping or an unpleasant stinging sensation in the calf during use of Relegs®. The intensity should then be reduced. Sometimes it can also help to place the electrode patch in a different position on the calf.
- Relegs® was not designed to diagnose or cure an illness. Please consult your physician first if you have RLS.

3.3. WHEN SHOULD RELEGS® NOT BE USED?

- The device should not be used simultaneously with any other type of medical electrical equipment.
- The device should not be used when driving a motor vehicle or operating machinery.
- The device should not be used whilst showering, swimming, bathing, in the sauna or in any other environment with a high humidity.
- The device should not be used near flammable substances and gases, or near explosives.

3.4. WHO SHOULD NOT USE RELEGS®?

- You must not use Relegs® in the following situations:
 - a) with a pacemaker, defibrillator or other electronic implant (transdermal release system for medication, pumps)
 - b) for 6 months after the diagnosis of deep vein thrombosis (DVT) has been made (can cause a pulmonary embolism and death).
- In the following situations, you should consult with your physician before using Relegs®: a) altered pain sensation in the legs b) with epileptic seizures
- c) with severe illness.
- Relegs® is not intended for use by people with psycho-emotional disorders or diminished mental capacity. They may not be able to understand the user manual properly and might become agitated by the treatment.

3.5. USE BY CHILDREN AND ADOLESCENTS

- Children should not be treated with Relegs®.
- Relegs® should be stored in such a way that children and adolescents under the age of 18 do not have access to it.
- Ensure that children cannot get their hands on Relegs®. Children can swallow small parts and this can cause choking. Children can injure themselves with the use of Relegs®.

3.6. USE OF RELEGS®

- Relegs® does not have an on/off switch and it works by tapping on it. It is best to tap the middle of the stimulator. The stimulator can be switched on or off by tapping on it twice. Tapping once (1x) will change the strength of the pulse (intensity).
- Relegs® and the electrode patches provided may only be used in that specific combination.
- The electrode patches are for personal use.
- Relegs® has been clinically tested to be attached to the calf via the electrode patch. People can experience the symptoms of RLS very differently. Place the electrode patch over the area where you experience the (most) symptoms. Some people have a trigger point, from which the symptoms develop each time. In that case, place the electrode patch on the trigger point.
- Apply the electrode patch with light finger pressure along the entire electrode patch, to achieve good contact with the skin.
- Each individual responds differently to electrical stimulation. Therefore, the placement of the electrode patch can differ from person to person. Place the electrode patch over the area of the calf/leg where you experience the best response and wherein feels most pleasant.
- If the intensity of the stimulation is too high or too low for you, you can simply adjust it yourself in the calibration position (see 8.1).
- The device should be switched off when you connect Relegs® to the electrode patch or when you disconnect it.
- If you want to reposition the electrode patch during use, then you should switch Relegs® off first.
- The adhesive strength of the electrode patch depends on the skin quality, the storage and the number of times that it is used. The electrode patch should be replaced for a new one if it no longer attaches properly to the skin along its entire surface.
- If the electrode patch no longer attaches properly, then you may be able to restore some of the adhesive strength by spreading a few drops of water over the patch (ensure that the electrode patch is disconnected from Relegs®).

- If the electrical stimulation results in a stinging sensation, then it can help to moisten the electrode patch with a few drops of water (ensure that the electrode patch is disconnected from Relegs®).
- Avoid touching the conducting hydrogel on the electrode patch during use.
- If the device malfunctions during use, or if you want to stop the treatment immediately, tap twice on the stimulator to stop it immediately.
- Under certain circumstances, the use of the electrode patches can cause skin problems. You should not use Relegs® any more and you should consult your physician.

7. WHERE SHOULD THE ELECTRODE PATCH NOT BE ATTACHED?

- Relegs® should not be attached to the following parts of the body:
 - a) on the chest and/or near the heart;
 - b) on wounds, eczema and/or skin inflammation;
 - c) on skin where you do not have normal sensation;
 - d) on the following parts of the body: skull, eyes/eyelids,
on or in the mouth, front of the neck, the larynx (voice box),
the throat, the carotid artery;
 - e) any area where internal bleeding has recently occurred
(this does not include a superficial bruise);
 - f) in the vicinity of a malignant tumor;
 - g) over the abdomen or the pelvis during pregnancy.

4. Contents of the packaging

Check the contents of the packaging prior to use, ensuring that all the items listed below are included and are not damaged. Please contact service@relegs.com if one of the items is - or appears to be - damaged.

- Relegs® device (stimulator and charger)
- USB/USB-C cable
- User manual

4.1. MAIN COMPONENTS

1. **Stimulator:** Sealed casing with indicator lights, two magnets buttons for connection of the electrode patch and two charging points.
2. **Electrode patch:** Patch on protective foil (a) with finger grips to grab hold of (b) and conductive hydrogel (c) to attach on the skin.
3. **Charger:** Casing with USB port, two metal studs for connection of the stimulator and two charging pins.

figure 1: Main components of Relegs®

5. Charging the battery

- When you have purchased Relegs®, you should first charge the battery. Charging will take Relegs® out of the storage setting. Relegs® is supplied with a USB cable and can only be charged via USB.
- Do not use Relegs® whilst charging.
- If you do not use the device for an extended period, you should charge the battery at least every 6 months to avoid excessive discharge.
- You can charge the batter by placing the stimulator in the charger. The charger needs to be connected to mains power via a USB-AC adapter or via the PC.
- Ensure that the stimulator is placed in the charger in the correct orientation. The charging pins on the charger need to touch the charging points on the stimulator.
- A green indicator light will start flashing on the stimulator once it starts charging; It can take up to a minute before the indicator light starts flashing if the battery is very empty. The indicator light will burn green continuously once the battery is fully charged. The green indicator light will switch off once the stimulator is removed from the charger.
- If the battery is almost empty, an orange indicator light will flash 6 times on the stimulator.
- For your safety, the battery should never be left unattended whilst charging.

5.1. SYSTEM REQUIREMENTS FOR CHARGING

THE BATTERY VIA USB

- USB-AC adapter.** Using a commercially available USB-AC adapter with an output of 1.5 A or 0.5 A.
- Personal Computer** with USB port.

6. Maintenance and Cleaning

6.1. MAINTENANCE

- Relegs® is maintenance-free.
- Do not dismantle or repair Relegs®, as this can result in technical accidents or physical injury. Warning! Danger to life!

6.2. CLEANING AND CARE

- Relegs® can be cleaned with a soft cloth that has been slightly moistened with water. Rub dry afterwards. A mild detergent can be used for larger stains. Do not use chemical detergents or abrasive products to clean Relegs®. Ensure that Relegs® is not switched on during cleaning.
- For reasons of hygiene, each user should use his or her own electrode patch.
- A suitable, commercially available disinfectant can be used to disinfect the device. Allow Relegs® to dry thoroughly immediately after disinfection.
- Do not immerse Relegs® on water or other liquids.

7. Disposal


7.1. DISPOSAL

- Used electrode patches can be disposed of as normal household garbage.
- Relegs® contains a lithium battery. Dispose of the device and the components in accordance with the relevant statutory regulations. Unlawful disposal can cause environmental pollution.
- The Management of White and Brown Goods Decree states that electronic products may not be treated as household garbage and may not simply be discarded. Dispose of your old Relegs® in accordance with the EU directive 2012/19/EU-AEEA regarding waste electrical and electronic equipment. Obtain the required information from your municipality or a waste disposal company.


8. Instructions for use of Relegs®

Use Relegs® on the leg where you experience the most symptoms!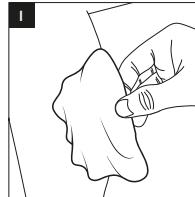


STEP I

Clean the area of skin thoroughly, so that it is free of oil and clean for attachment of the electrode patch.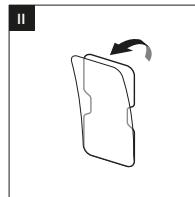

STEP II

Remove the protective foil from the electrode patch.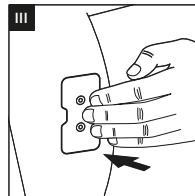


STEP III

Place the electrode patch on the calf. Ensure that your fingers do not touch the conductive hydrogel in the process. Use the finger grips along the edges of the electrode patch.

STEP IV

Connect the stimulator to the electrode patch using the magnetic buttons. **Caution!** Relegs® should not be switched on when attaching the electrode patch!

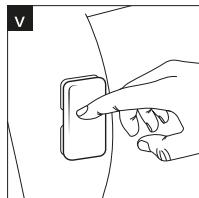STEP V

Switch the stimulator on by tapping on it twice. Relegs® has 4 settings and always starts at the lowest setting. To switch to a higher setting, simply tap the stimulator once. Tapping the device whilst in setting 4 will automatically return the device to the lowest setting.

A green flashing indicator light shows you which setting you are using. Select the highest acceptable intensity. This should at least produce a strong, but still comfortable, sensation.

STEP VI

Relegs® has a pre-set treatment time of 30 minutes and switches off automatically after this period. If you want to stop using the device before the end of the time, simply tap on the stimulator twice. An orange light will flash once. This can be done from any setting.

STEP VII

After use, disconnect the stimulator from the electrode patch and carefully remove the electrode patch from the skin. Place the electrode patch back on the protective foil and store it in the resealable plastic bag to prevent dehydration. Ensure that your fingers do not touch the conductive hydrogel in the process. Use the finger grips along the edges of the electrode patch. **Caution!** Never remove the electrode patch from the skin together with Relegs®, particularly if the device is still switched on!

8.1. CALIBRATION SETTING / ADJUSTING THE INTENSITY OF STIMULATION

- Available from series number starting with 21 and higher. If the intensity of the stimulation is too high or too low for you? Then you can easily adjust this yourself! With the stimulator switched on (regardless of which setting is being used), tap on the stimulator three times. The green indicator light will start to burn continuously and the stimulator is now in the calibration setting, meaning that the intensity will increase automatically every 2 seconds. You can now select the intensity that suits you best by tapping once (1x) on the stimulator. The intensity that you have selected will now become the middle value for 4 new intensity settings.
- N.B.! If you have selected an option by tapping once on the stimulator, then the stimulator will switch off automatically. You will then need to switch the stimulator on first (tap 2x) before continuing. The same applies if you have not selected a new setting and the highest intensity in the calibration mode has been reached! Relegs® will then select the highest intensity as your chosen intensity.

9. Explanation of indicator lights

INDICATOR LIGHT	EXPLANATION
Orange light flashing (6x)	Battery almost empty; Relegs® will switch off automatically.
Orange light burns for several seconds	Stimulator detects poor connection between the electrode patch and the skin; Relegs® will switch off automatically.
Green light flashing 1, 2, 3 or 4 times	Indicator, reflects intensity setting.
Green light flashing in charger	The battery is charging.
Green light burning continuously in charger	The battery is fully charged.
Green light burning continuously (stimulator not in charger)	The stimulator has switched to the calibration mode

10. Technical malfunction

MALFUNCTION	CAUSE	SOLUTION
Relegs® does not switch on.	The battery does not have enough power.	Charge the battery.
	Relegs® detects poor contact between the electrode patch and the skin.	Press the electrode patch onto the skin.
	The electrode patch is worn out and needs replacing.	
Relegs® stops working whilst in use.	Relegs® has accidentally been switched off.	Switch the device on by tapping it twice (2x).
	The battery does not have enough power.	Charge the battery.
	Relegs® detects poor contact between the electrode patch and the skin.	Press the electrode patch onto the skin.
		The electrode patch is worn out and needs replacing with a new one.

11. Electromagnetic immunity

Relegs® is suitable for use at home and in a clinical setting. With regard to EMC and electrical safety in the intended operational environment, it has been demonstrated that the device meets all applicable requirements of IEC 60601-1, IEC 60601-2, IEC 60601-2-10 and IEC 60601-1-11.

12. Technical specifications

GENERAL INFORMATION	
Model type	Relegs®
Dimensions of stimulator (lxbxh)	approx. 75x45x12 mm
Weight of stimulator (incl. batt.)	approx. 35 grams
Surface area of electrode patch	1140 mm ²
Material	Plastics (bio-compatible), Metals
Essential performance	With normal use, Relegs® emits safe electrical pulses in the calf. Any malfunctioning of Relegs® does not pose a hazard to the user and/or environment.
	Manufacturer: Relegs B.V. Kapteynstraat 1 2201 BB Noordwijk, The Netherlands
Production date	YYYY-MM-DD (year, month, day) 
IP-coding	Stimulator IP22 Charger IP21
SN	Serial number

ELECTRICAL INFORMATION	
Electricity supply	Lithium battery
Voltage	0-45 Volt at 500 Ohm load
Pulse shape	Symmetrical biphasic
Frequency	54-81 Hz (vibrations per second)
Pulse duration	200 µs (microseconds)
Pulse strength	0-50 mA (milliampere)
Electrical tolerances	+/- 20% at 500 Ohm load
USAGE INFORMATION	
Temperature of surroundings	5°C to 40°C (degrees Celsius)
Max. humidity with normal use	15 % - 90 % (percent)
Atmospheric pressure	700 hpa – 1060 hpa (hectopascal)
Lifespan of Relegs®	Device with battery should last at least 2 years.
Expected lifespan of the electrode patch	Unopened, stored in resealable plastic bag: 1 year. Once opened, max. 1 month. Use max. 10 times.
STORAGE / TRANSPORT INFORMATION	
Relegs® will remain operational following storage/transport in the following environment: -25 °C to +5°C (degrees Celsius) and +5 °C to 35°C at a relative humidity up to 90% (percent), non-condensing; +35 °C to 70°C at a water vapor pressure up to 50 hPa (hectopascal).	
Prolonged storage of electrode patch between 5°C and 27°C; short periods up to 4 weeks at 0°C to 40°C.	

13. Limited warranty

The Relegs® device that you have purchased was developed and manufactured with the utmost care. The statutory warranty period is 24 months from the date of purchase, covering material and manufacturing errors of the product. Please retain your sales receipt as proof of purchase of the Relegs® device to make a valid claim on the warranty.

The warranty does not cover:

- Damage resulting from incorrect use;
- Defects that the customer was already aware of upon purchase;
- Damage resulting from unauthorised interventions and through the customer's own fault;
- Electrode patches are excluded from the warranty once the resealable plastic bag has been opened.

Please contact us in the following manner in the event of technical problems, questions or warranty claims relating to Relegs®:

INSTRUCTIONS:

If you have a complaint about Relegs®, please first contact the service center via service@relegs.com or via www.relegs.com. You will receive instructions for returning the device after authorization by the service center; never return a device without authorization.

Relegs B.V. will replace devices if it is proven that they do not function according to the warranty stated above, which fall under this limited warranty and which are returned with sufficient postage to Relegs B.V. Please contact us at service@relegs.com or via www.relegs.com to initiate a return.

The original warranty will not be extended if Relegs B.V. replaces a device.

RELEGS BV
Kapteynstraat 1
2201 BB Noordwijk,
Nederland /
Die Niederlande /
the Netherlands
service@relegs.com

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Fabricant : l'appareil Relegs B.V. Kapteynstraat 1 2201 BB Noordwijk, Pays-Bas
Dans le cas où Relegs B.V. remplacerait un appareil, la garantie initiale ne sera pas prolongée.
RELEGS B.V.